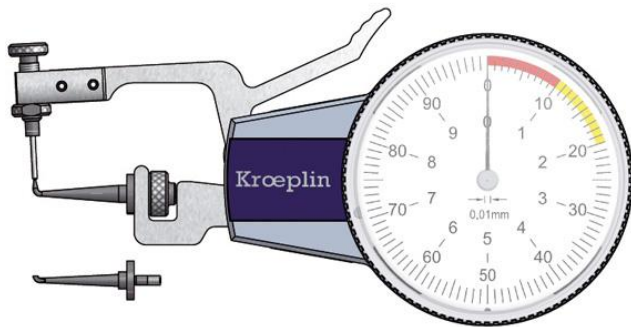
- D** Bedienungsanleitung
- GB** Operation instructions
- F** Mode d'emploi

E110D • POCO2D • E110W

E110D

Messbereich / measuring range / Plage d'affichage: 0 – 2mm

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*Subject to technical alterations.
If because of this instruction, rules in the
Product Liability Act are referred to in
case of damage these will herewith be
explicitly excluded.*

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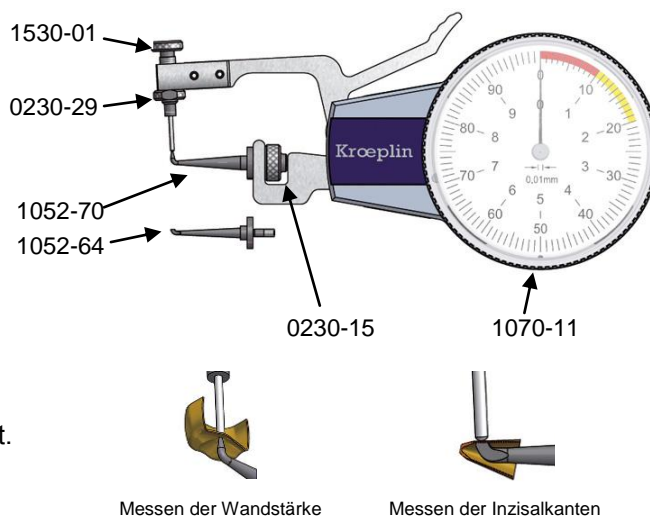
The standard supply of the gauge E110D includes the lower caliper arm no. 1052-70 for the measurement of wall thicknesses and fissures. This arm is assembled and adjusted. Another caliper arm for the measurement of incisal margins, no. 1052-64, is also included, but not assembled.

Changing the caliper arm:

1. Loosen the knurled nut 0230-15 and remove the caliper arm.
2. Insert the other arm in the base and tighten the knurled nut. Please see to it that the pin at the base fits exactly into the groove of the caliper arm.
3. Adjust the instrument:
 - Loosen the knurled nut no. 0230-29 at the upper measuring contact.
 - Turn the upper measuring contact no. 1530-01 so that both pointers indicate approximately the zero position.
 - Tighten the knurled nut no. 0230-29
 - Turn the scale until the big pointer indicates the zero position. The small pointer does not have to indicate exactly the zero position.

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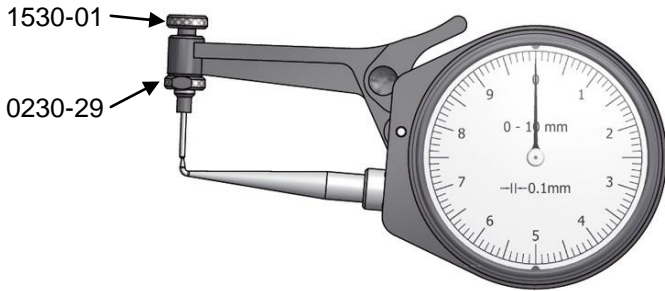
The gauge POCO2D is used for the measurement of wall and base thicknesses of crowns and fissures.

Renew the measuring contact 1530-01:

1. Loosen the knurled nut no. 0230-29 and remove the measuring contact.
2. Insert the new contact and tighten the knurled nut.
3. Adjust the instrument:
 - Loosen the knurled nut no. 0230-29 at the upper measuring contact.
 - Turn the upper measuring contact no. 1530-01 so that the pointer indicates the zero position.
 - Tighten the knurled nut no. 0230-29.

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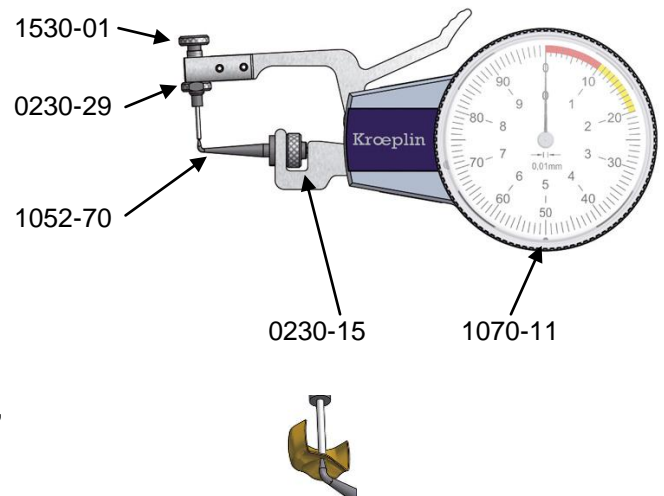
The gauge E110W is used for the measurement of wax thicknesses. It has a very smooth mechanism and a small measuring pressure.

Renew the caliper arm 1052-70:

1. Loosen the knurled nut 0230-15 and remove the caliper arm.
2. Insert the other arm in the base and tighten the knurled nut. Please see to it that the pin at the base fits exactly into the groove of the caliper arm.
3. Adjust the instrument:
 - Loosen the knurled nut no. 0230-29 at the upper measuring contact.
 - Turn the upper measuring contact no. 1530-01 so that both pointers indicate approximately the zero position.
 - Tighten the knurled nut no. 0230-29
 - Turn the scale until the big pointer indicates the zero position. The small pointer does not have to indicate exactly the zero position.

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Spare parts and accessories

knurled nut
turnable scale
caliper arm for the measurement of thicknesses
measuring contact

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